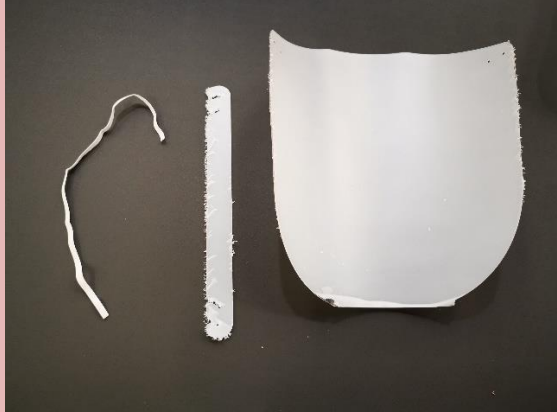

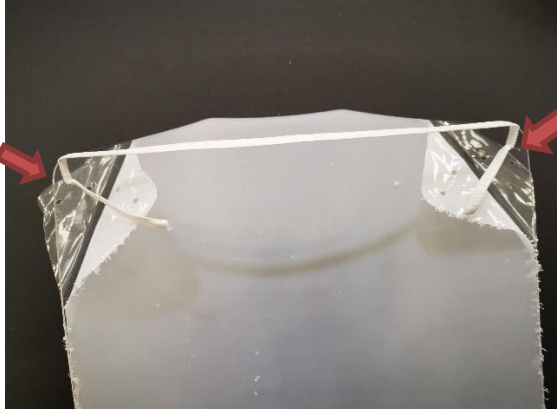






## Donite Face Shield – Assembly

Recommended P.P.E for assembly - Wear gloves, respirator mask and face shield.		STEP
	Contents required: shield, headrest and elasticated band.	
	Remove the protective coating on both sides of the shield exposing the four holes.	1
	Pass the elasticated band in through the outer holes.	2

	<p>Loop the ends of the elasticated band back through the Shield using the inner holes.</p>	<p>3</p>
	<p>Remove the protective sheeting on both sides of the headrest</p>	<p>4</p>
	<p>Slot the headrest in through the inner loops of the elasticated band.</p>	<p>5</p>
	<p>Adjust band to fit securely</p>	<p>6</p>
	<p>Once product is ready to be used, only then remove protective layers to ensure the shield is sterile.</p>	<p>7</p>

If you would prefer to watch an assembly video, please visit

<https://www.donite.com/covid-19-protection.php>